


Listo

User Guide

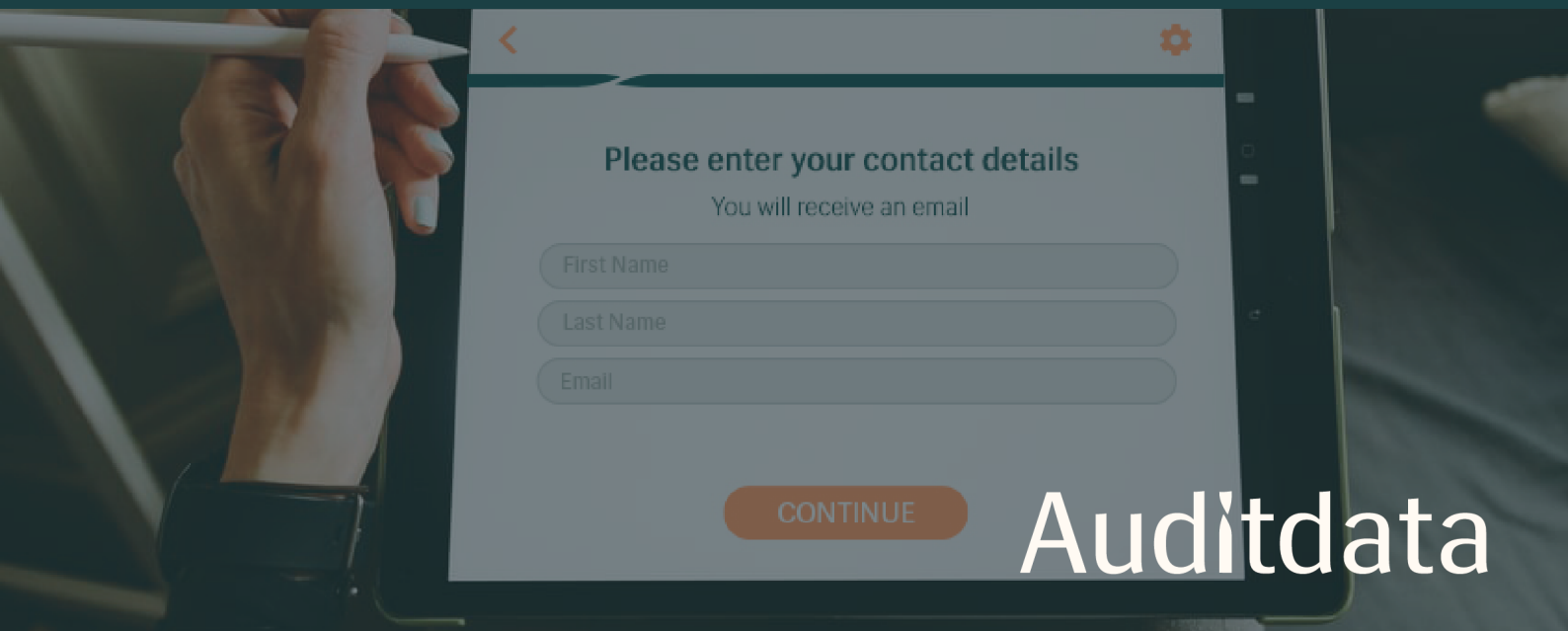

Auditdata


1 SET UP AND GET STARTED	3
2 MANAGE TEST FLOWS	9
2.1 Create a Test Flow	9
2.2 Customize a Test Flow	14
2.3 Types of Test Flow Pages	17
3 RUN HEARING TESTS	27
4 GET LISTO REPORTS	33

1 Set Up and Get Started

Cloud Administration System allows for Listo application setup and management. With a number of configuration options, you can adapt Listo to your current needs.

To start administering your app:

Log into **Auditdata Cloud** > click **Screener** > select **Administration**

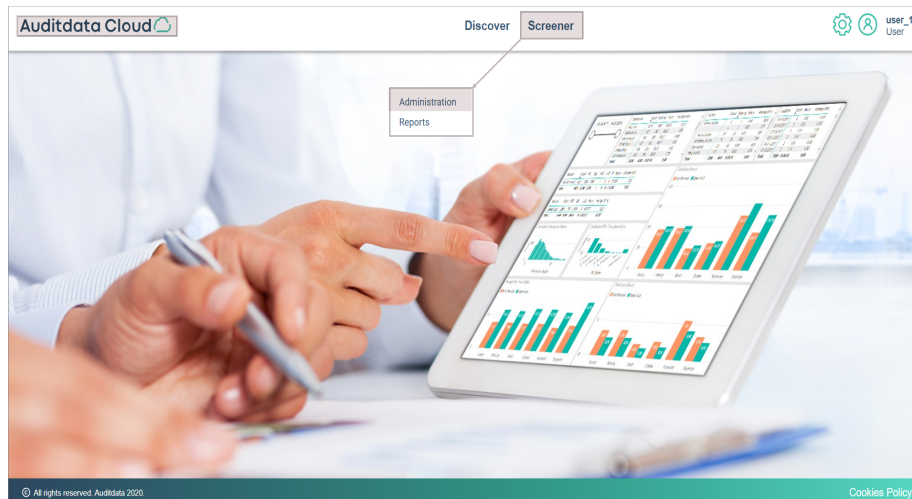


All **Administration tasks** in the Cloud are grouped under the following tabs:

- Offices
- Test Flows
- Resources
- Styles
- Settings
- Translation
- Email Templates
- Email Resources

OFFICES


The *Offices tab* enables you to automatically import offices from Aurora. To get the list of available offices, employ the **SYCNHRONIZE** option:

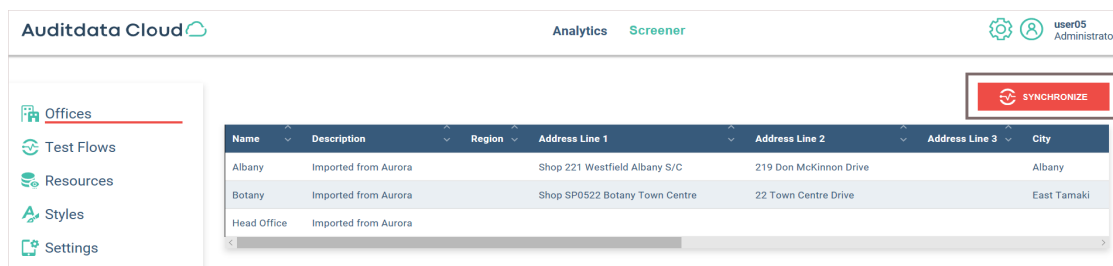

TEST FLOWS

The *Test Flows* tab can be used for building and customizing Test Flows for hearing assessment.

- To add a new Test Flow, click the **ADD FLOW** button.
- To upload a new Test Flow, click **UPLOAD FLOW** and drop files into the box.
- To **export**, **add pages** or **delete** an existing Flow, make use of the respective controls on the right.

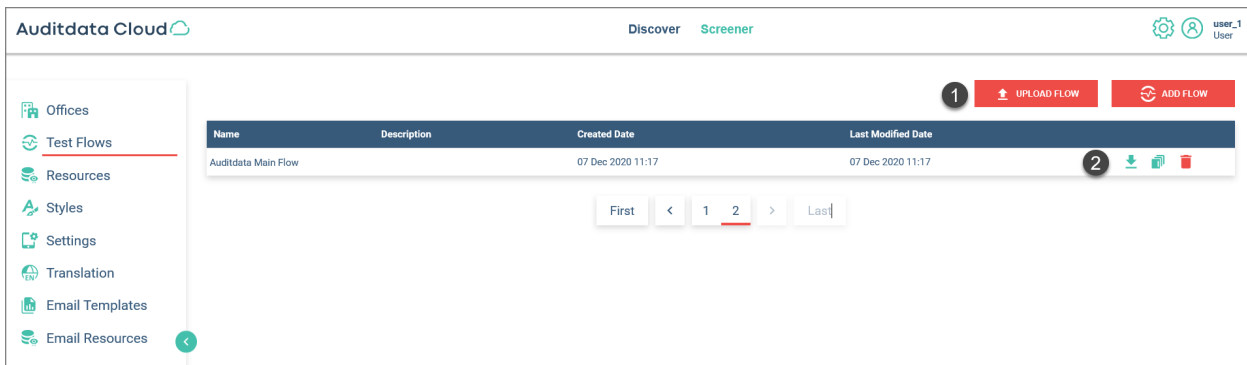


RESOURCES

The *Resources* tab represents a content folder for images and videos that can be used when building or modifying Test Flow pages.

You can add customized background images to all of your pages, whereas video files are restricted to *Video* and *Result* pages. For more information, refer to [Section 2.3: Types of Test Flow Pages](#).

Thumbnail previews are available for all uploaded images. Clicking a thumbnail opens an enlarged image view.

- Use **UPLOAD** to place images and videos in the Resources repository.
- Optionally, **delete** irrelevant resources or **download** selected media files to your computer.

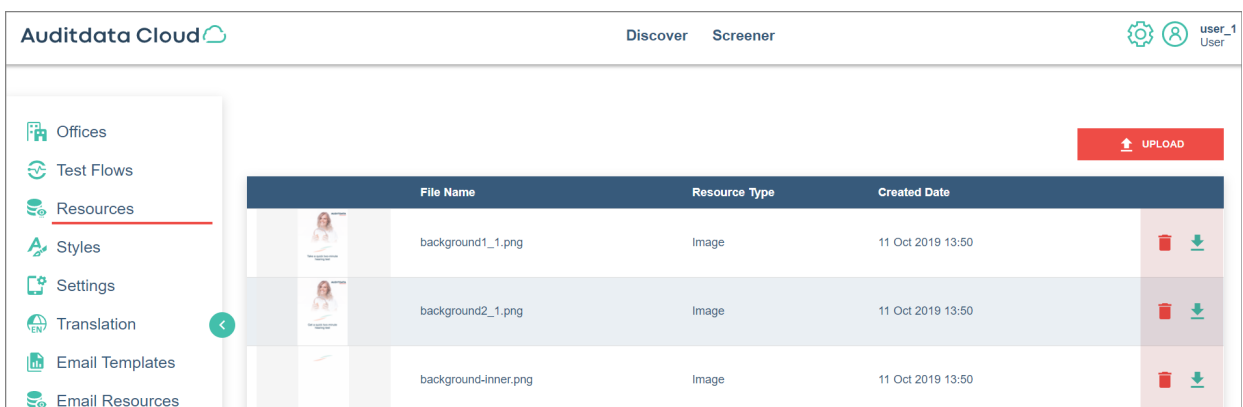

MEDIA FILE SPECIFICATIONS

To ensure good quality of background images and video playback, follow the guidelines below.

Parameters	Background Image	Video
<i>Supported formats</i>	.jpg, .jpeg, .png	.mp4, .mov
<i>Recommended size</i>	500 KB	10 MB
<i>Recommended resolution</i>	1536 x 2048 pixels 4:3 ratio (~264 ppi density)	640 x 480 pixels (30 fps frame rate)
<i>Bitrate</i>	not applicable	2.5 Mbps

STYLES

The *Styles tab* makes it possible to set defaults for your Listo app Test Flows. For example, you can specify a background image, text font, size and color, response button color, page title font size, etc.

Defaults are automatically applied to the added **Test Flow** pages unless you decide to make your own customizations.

To configure a default:

- Enter relevant values in the provided field and then click **Save** at the bottom of the page.

The screenshot displays the 'Styles' configuration interface. On the left, a sidebar lists navigation options: Offices, Test Flows, Resources, Styles (highlighted), Settings, Translation, Email Templates, and Email Resources. The main area contains the following settings:


- Background colour: #ebebfb
- Background image: (dropdown menu)
- Navigation bar background colour: #ffffff
- Button background colour: #417505
- Pressed button background colour: #a32929
- Button text colour: #ffffff
- Button font size: 40
- Button height: 80
- Button width: 300
- Response button background colour: #ef4c47
- Response pressed button background colour: #bb413f
- Response button text colour: #ffffff
- Response button font size: 50
- Page title font size: 30
- Description font size: 20
- Font size for result text: 30
- Logo height: 50
- Logo width: 100
- Text colour: #000000
- Progress bar colour: #342d2c
- Text input colour: #1b1a1a
- Text input background colour: #e4e5e9

A red 'SAVE' button is located at the bottom center of the configuration area.

SETTINGS

The *Settings* tab includes security-related functionality:


- Setup password
- Noise correction password
- Domains blacklist (usage of blacklisted emails is not allowed in Listo)

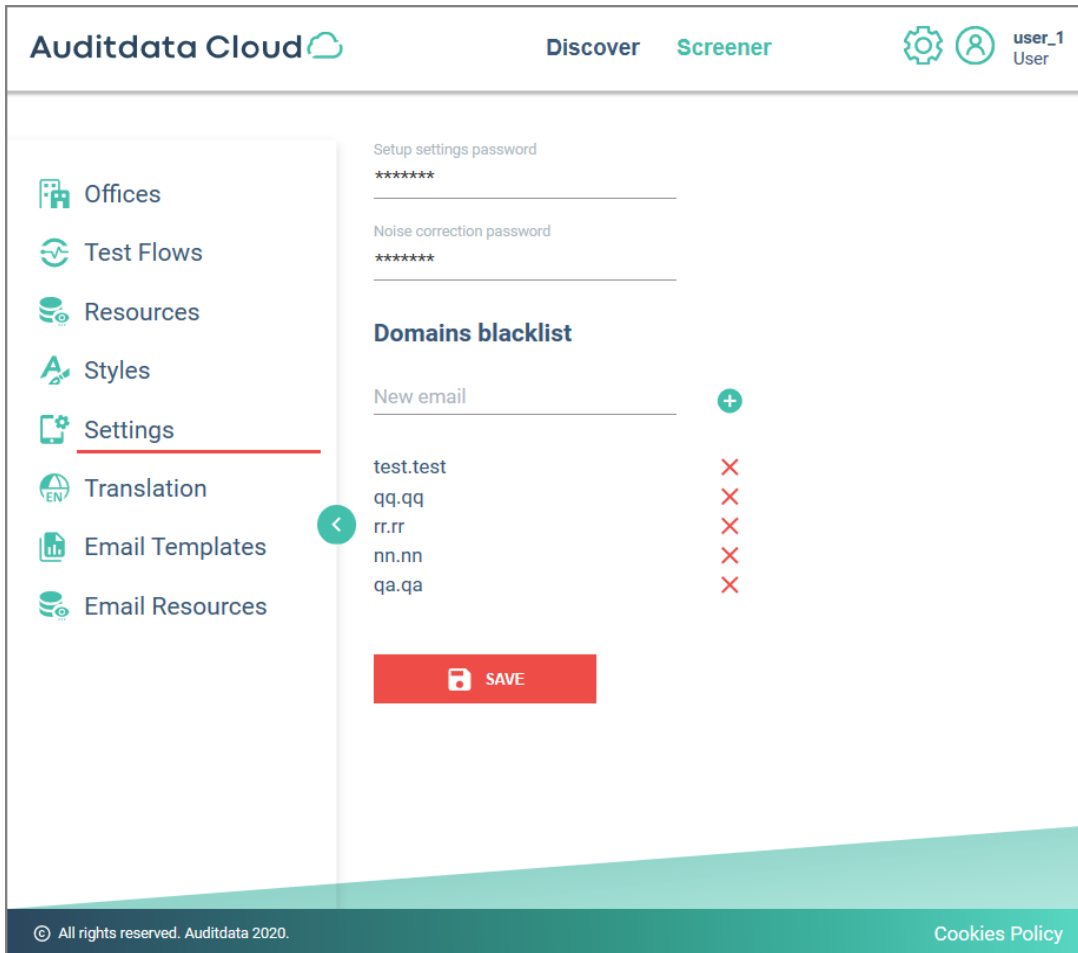

TRANSLATION


The *Translation* tab makes it possible to set customized names for the default tabs:

- Months
- Password
- Update
- Noise Level
- Setup Application
- Error

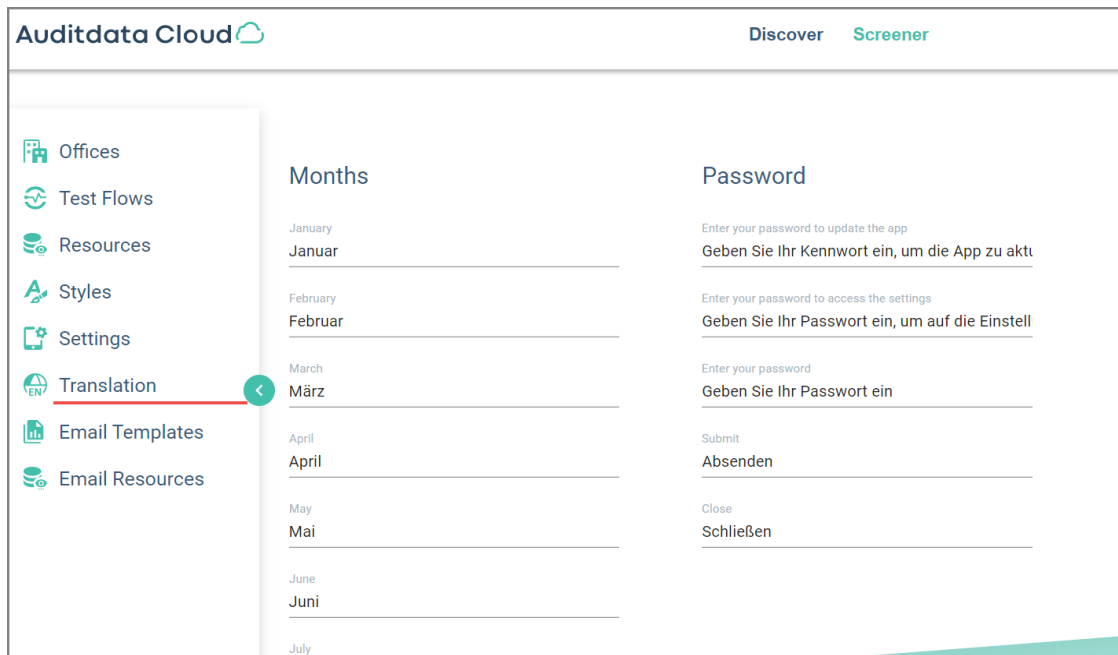


EMAIL TEMPLATES

The *Email Templates Tab* is a content folder for templates that can be used to create a generic format for your email letters. To **add** a template, use the corresponding button. To **modify** or **delete**, employ controls on the right.

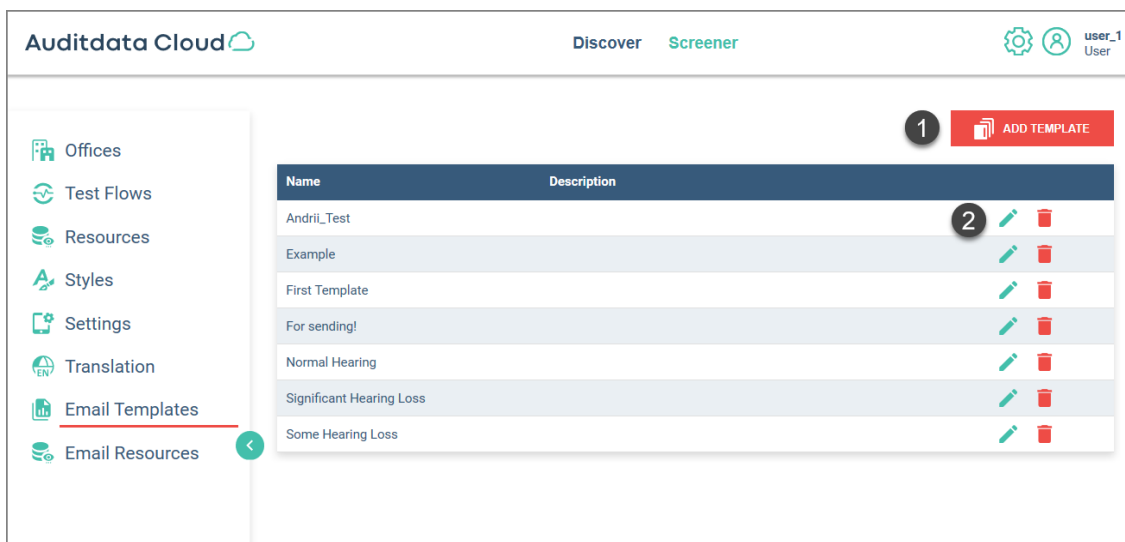


When adding a template, you can specify the template's *name*, provide a *description* and *subject text* to make it more distinguishable and easy to find. All available template settings can be adjusted in the built-in editor.

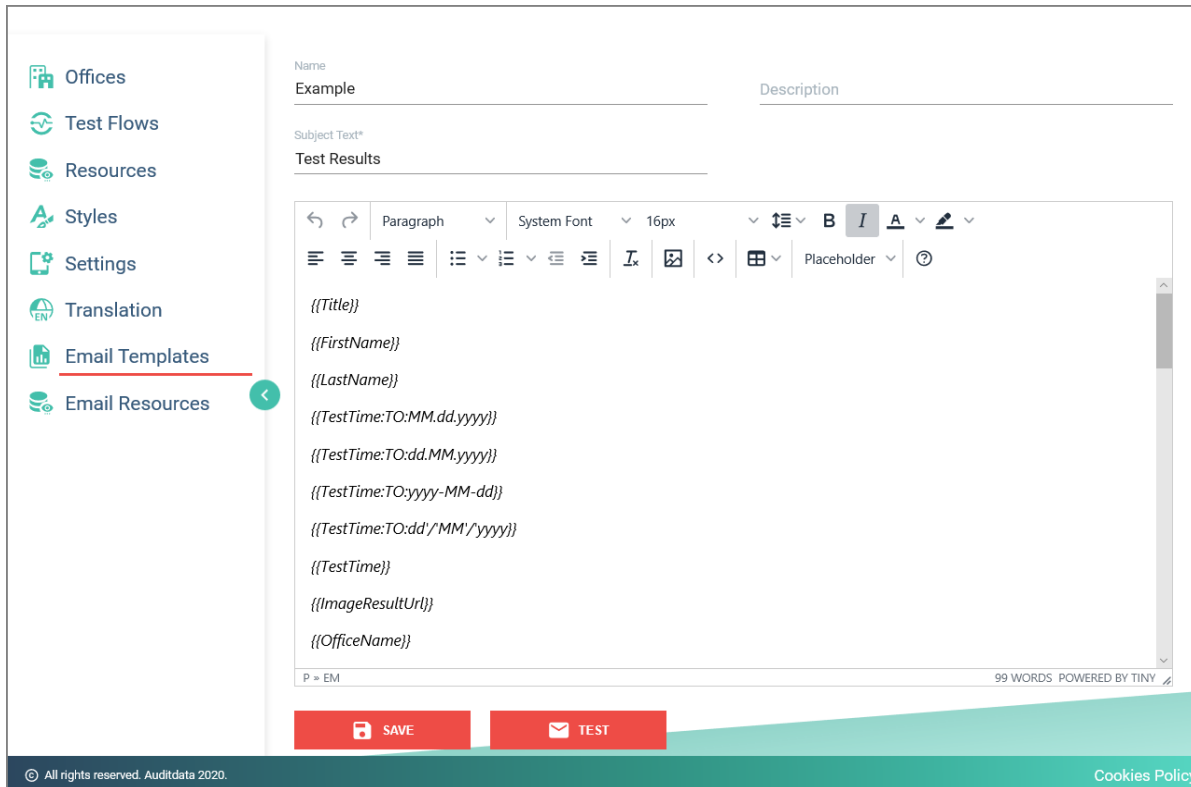

When you are finished creating or changing your template, click **Save** at the bottom of the page. If required, you can optionally **TEST** your template before use.

EMAIL RESOURCES

The *Email Resources* tab is a content folder for images and videos that can be used when building or modifying Email Templates. Thumbnail previews for all uploaded images will be shown in the list. Clicking a thumbnail opens a larger view.

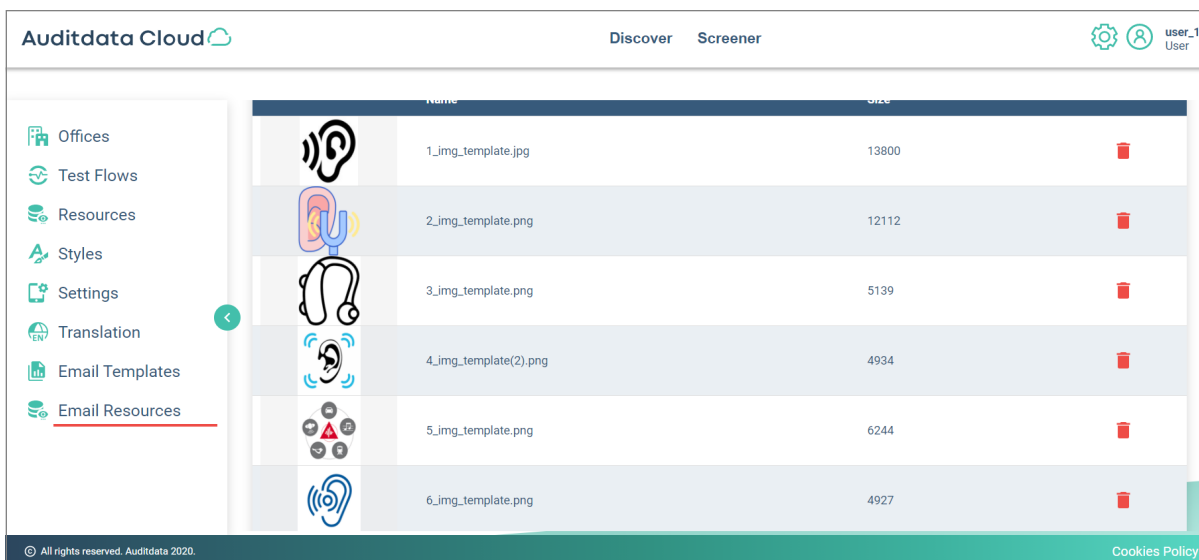

2 Manage Test Flows

2.1 Create a Test Flow

Despite having some default Test Flows, you may need to create a Test Flow that is tailored to a specific use case.

Since all *Listo Test Flows* are managed from the *Cloud Administration System*, you need to:

1. Log into **Auditdata Cloud** > click **Screeener** > select **Administration**
2. From the Navigation Pane on the left, select **Test Flows**
 - To add a Flow manually, use the **ADD FLOW** button
 - To upload an already configured flow, click **UPLOAD FLOW**

Name	Description	Created Date	Last Modified Date	
Auditdata 2019 Main Flow		21 Oct 2020 11:28	09 Dec 2020 23:12	
Customized Screening Flow		09 Dec 2020 16:59	09 Dec 2020 23:30	
Flow with Online Booking		23 Oct 2020 09:20	09 Dec 2020 23:13	
Listo Sample Hearing Test		30 Nov 2020 08:36	09 Dec 2020 23:23	
My Basic Flow		02 Sep 2019 11:51	09 Dec 2020 23:16	
New Quick Hearing Test		30 Nov 2020 08:36	09 Dec 2020 23:20	
Official Demo Flow 2019		21 Oct 2020 11:28	09 Dec 2020 23:19	
Smart Audiometry Test		23 Oct 2020 09:20	09 Dec 2020 23:26	

If you want to proceed with the manual flow creation, follow the steps below.

3. In the pop-up window, enter a **Name** and optionally a **Description** for your flow.

Add New Test Flow

Name

Auditdata 2020 Demo Flow


Description

CLOSE **SAVE**

4. The added flow will be marked "New" and displayed in the list
- Use the **pages** (📄) icon to further configure your flow

Name	Description	Created Date	Last Modified Date	
Auditdata 2019 Main Flow		21 Oct 2020 11:28	09 Dec 2020 23:12	📄 🗑️ 🔄
Auditdata 2020 Demo Flow		09 Dec 2020 23:41	09 Dec 2020 23:41	📄 🗑️ 🔄
Customized Screening Flow		09 Dec 2020 16:59	09 Dec 2020 23:30	📄 🗑️ 🔄
Flow with Online Booking		23 Oct 2020 09:20	09 Dec 2020 23:13	📄 🗑️ 🔄
Listo Sample Hearing Test		30 Nov 2020 08:36	09 Dec 2020 23:23	📄 🗑️ 🔄
My Basic Flow		02 Sep 2019 11:51	09 Dec 2020 23:16	📄 🗑️ 🔄
New Quick Hearing Test		30 Nov 2020 08:36	09 Dec 2020 23:20	📄 🗑️ 🔄
Official Demo Flow 2019		21 Oct 2020 11:28	09 Dec 2020 23:19	📄 🗑️ 🔄
Smart Audiometry Test		23 Oct 2020 09:20	09 Dec 2020 23:26	📄 🗑️ 🔄

5. On the Test Flow configuration page, click **Add Page** to start building your Flow
- In the "Add Page" pop-up, enter the name and, if required, a description of the page, and select the Page Type from the drop-down list
 - **Save** to proceed

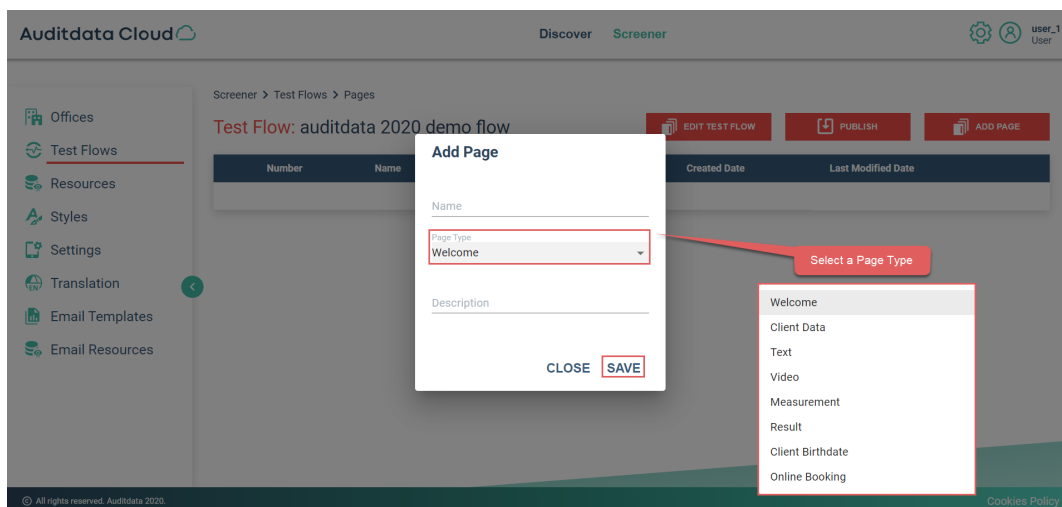

Your Test Flow must have at least 3 pages:

- Welcome
- Measurement
- Result


These pages are mandatory. The *Welcome* page must always come first and the *Result* page must be placed after the *Measurement* page. Otherwise, you will not be able to save your flow and a warning message will be displayed.

You can add a maximum of 8 pages: only one page of the same type is allowed per flow.

- Having added all the necessary pages, you can make adjustments to them by adding custom background images and button captions (*applicable to pages that have **continue** or **response-type** buttons*). Otherwise, these pages will use the defaults.
 - To **modify** a page, find that page in the list and click edit (✎) in the corresponding field.

If you have added a **Video page** to a Test Flow, you need to manually add a video file to that page because newly created video pages contain no default videos.

- To add a video, go into page *Edit* mode by clicking edit (✎).

- Having added all the required pages, you can adjust their order by using the  controls located to the left side of the list of pages. Move your pages up and down the list as needed.

The flow must be then **published** to become available on your iPad.

IMPORTANT: Unpublished flows will not be accessible in your Listo app.

Auditdata Cloud Discover Screener user_1 User


Screener > Test Flows > Pages

Test Flow: auditdata 2020 demo flow

EDIT TEST FLOW PUBLISH ADD PAGE

Number	Name	Type	Description	Created Date	Last Modified Date
1	Welcome			10 Dec 2020 00:53	10 Dec 2020 00:53
2	Client Data			10 Dec 2020 00:53	10 Dec 2020 00:53
3	Text			10 Dec 2020 00:54	10 Dec 2020 00:54
4	Video			10 Dec 2020 00:54	10 Dec 2020 00:54
5	Measurement			10 Dec 2020 00:54	10 Dec 2020 00:54
6	Result			10 Dec 2020 00:54	10 Dec 2020 00:54
7	Client Birthdate			10 Dec 2020 00:54	10 Dec 2020 00:54
8	Online Booking			10 Dec 2020 00:54	10 Dec 2020 00:54

8. After the flow is finalized and **published**:

- All pages will be marked with  (the view icon) signifying that the flow is in *View mode*.
- To go back to the *Edit mode*, click the **EDIT** button in the top right corner.
- You can also select email templates to be used with your flow. To choose templates, click **EMAIL SETTINGS** at the top. For more information on email templates and resources, refer to [Section 1. Set up and Get Started](#).

Auditdata Cloud Discover Screener user_1 User


Screener > Test Flows > Pages

Test Flow: auditdata 2020 demo flow

EMAIL SETTINGS VIEW TEST FLOW EDIT


Number	Name	Type	Description	Created Date	Last Modified Date
1	Welcome			10 Dec 2020 00:53	10 Dec 2020 00:53
2	Client Data			10 Dec 2020 00:53	10 Dec 2020 00:53
3	Text			10 Dec 2020 00:54	10 Dec 2020 00:54
4	Video			10 Dec 2020 00:54	10 Dec 2020 00:54
5	Measurement			10 Dec 2020 00:54	10 Dec 2020 00:54
6	Result			10 Dec 2020 00:54	10 Dec 2020 00:54
7	Client Birthdate			10 Dec 2020 00:54	10 Dec 2020 00:54
8	Online Booking			10 Dec 2020 00:54	10 Dec 2020 00:54

9. When you go back to the Test flow list, your flow will have a new icon in the right (). It means that this flow can now be **copied** and serve as a template to create new flows.

- Any existing flow can be reused or shared. Employ the **export** icon () to download your flow as a *.zip file*.

10. To be able to use the flow you have configured:

- Log into Listo
- Select your Test Flow
- Apply Flow

 Logout

SETUP APPLICATION

Selected office: Store Name

Choose an Office

Choose a Transducer

DD65

DT770Pro

HD622F

K72

Choose a Flow

Auditdata 2020 Demo Flow

Choose a Noise Level

Silent

Some Noise

A Lot of Noise



Apply the Flow


Version 1.3.2 License: 577yeMmzolsW









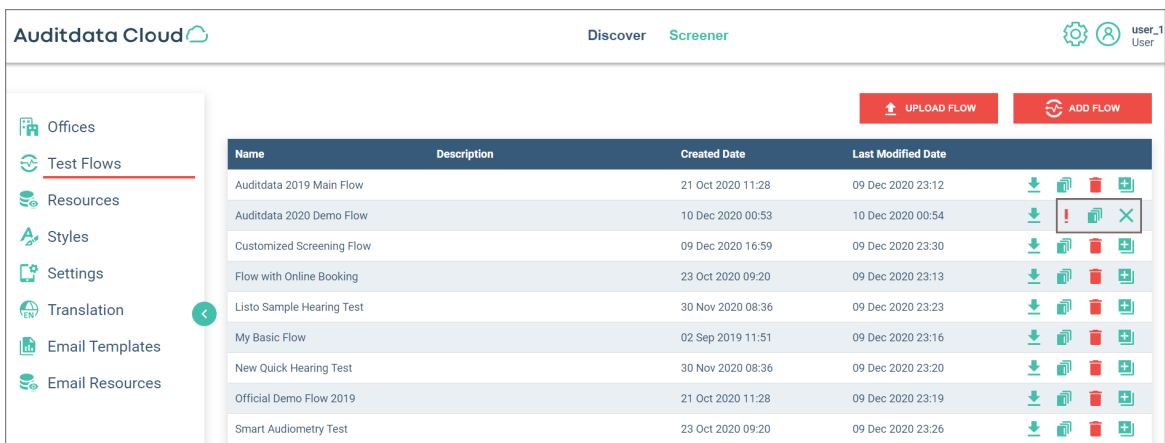




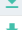

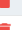





















2.2 Customize a Test Flow

Whenever required, you can make quick adjustments to your Test Flow. However, note that all changes have to be published.


To modify a Test Flow, you need to access **Auditdata Cloud > Screener > Administration**


1. In the left-side Navigation Pane, select the **Test Flows** tab to display the list of available Test flows
2. Find the **Test Flow** you want to change
3. To start **editing**, click the icon . To discard changes, employ .









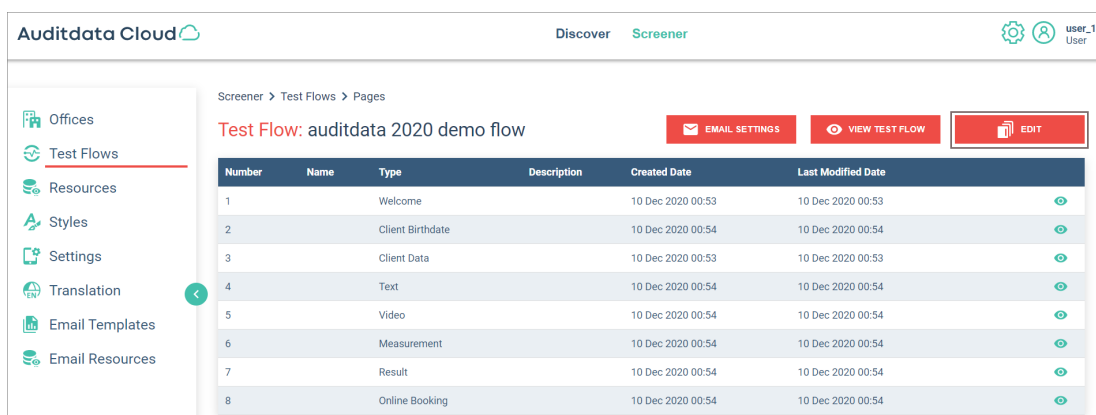

Name	Description	Created Date	Last Modified Date	
Auditdata 2019 Main Flow		21 Oct 2020 11:28	09 Dec 2020 23:12	   
Auditdata 2020 Demo Flow		10 Dec 2020 00:53	10 Dec 2020 00:54	    
Customized Screening Flow		09 Dec 2020 16:59	09 Dec 2020 23:30	   
Flow with Online Booking		23 Oct 2020 09:20	09 Dec 2020 23:13	   
Listo Sample Hearing Test		30 Nov 2019 08:36	09 Dec 2020 23:23	   
My Basic Flow		02 Sep 2019 11:51	09 Dec 2020 23:16	   
New Quick Hearing Test		30 Nov 2020 08:36	09 Dec 2020 23:20	   
Official Demo Flow 2019		21 Oct 2020 11:28	09 Dec 2020 23:19	   
Smart Audiometry Test		23 Oct 2020 09:20	09 Dec 2020 23:26	   


NOTE: if a flow is unpublished or contains any unpublished changes, an Alert (!) icon will be displayed.

4. The edit icon () will take you to the *Test Flow configuration page* where you can make all required changes. To start editing, click **Edit**.



Number	Name	Type	Description	Created Date	Last Modified Date	
1	Welcome			10 Dec 2020 00:53	10 Dec 2020 00:53	
2	Client Birthdate			10 Dec 2020 00:54	10 Dec 2020 00:54	
3	Client Data			10 Dec 2020 00:53	10 Dec 2020 00:53	
4	Text			10 Dec 2020 00:54	10 Dec 2020 00:54	
5	Video			10 Dec 2020 00:54	10 Dec 2020 00:54	
6	Measurement			10 Dec 2020 00:54	10 Dec 2020 00:54	
7	Result			10 Dec 2020 00:54	10 Dec 2020 00:54	
8	Online Booking			10 Dec 2020 00:54	10 Dec 2020 00:54	

5. The Flow will come into *Edit mode* with a number of new setup options appearing:
- **Edit test flow:** use it to change the flow's name, screensaver and margins
 - **Discard** any changes that should not be in the published flow
 - If the flow is finalized, you can use the **Publish** option
 - The **Add page** setting can only be used if a flow contains less than 8 pages
 - You can also modify (✎), delete (🗑) or change the order (⋮) of the Test Flow pages

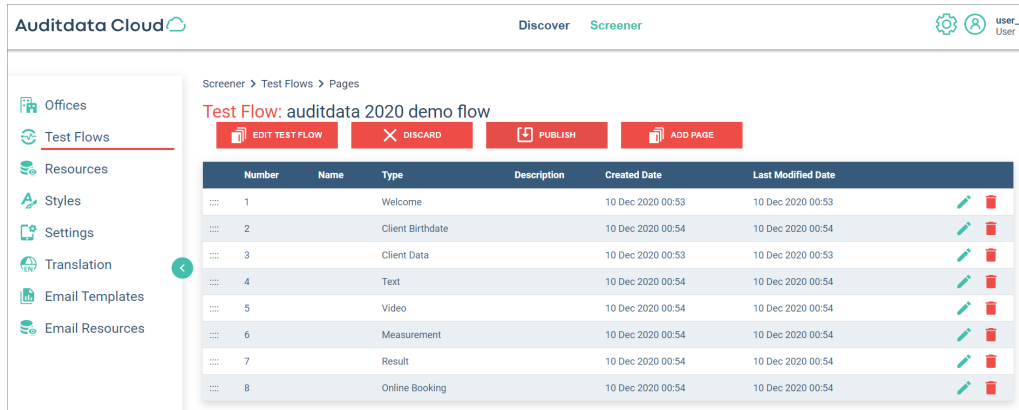

6. Click **Edit** (✎) to be able to change the page settings. In the *Edit window* that opens you can modify:

- The Name of the page
- Page Description
- The background image
- Page text or header text
- Button captions for pages that have buttons
- Add or replace a video file for Video pages
- Page content margins

Edit Welcome page

GENERAL
MARGINS

Name Description

Background Image ▼

Welcome Page Text
Welcome


Next Button Caption
Continue

CLOSE SAVE

All pages have different settings that can be changed. To learn more, refer to [Section 2.3: Types of Test Flow Pages](#).

- To be able to see your updates, open the Listo app on your iPad
 - An **Alert notification** will show that the changes have been made
 - Tap the *Alert icon* to **download** the update

The update will contain all published changes to existing Test Flows as well as any new Test Flows that have been published in the Auditdata Cloud.

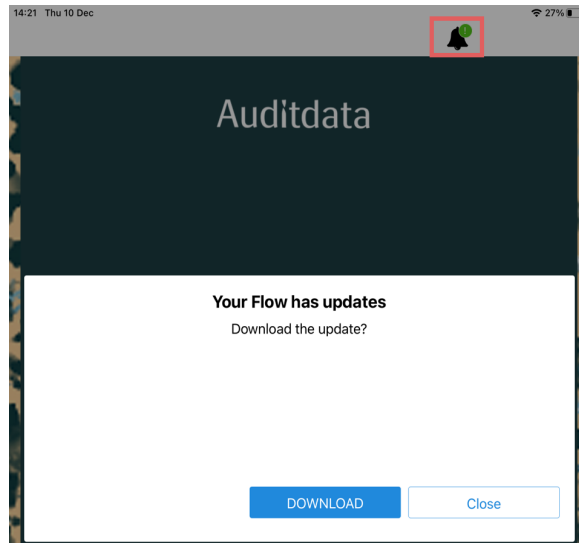

2.3 Types of Test Flow Pages


Auditdata Cloud Administration system offers a number of options for Test Flow configuration. Every Test Flow consists of 3 to 8 pages, with each page having different settings.

These are the types that can be added to a Test Flow:

1. Welcome
2. Client Data
3. Text
4. Video
5. Measurement
6. Result
7. Client Birthdate

The *Add page pop-up window* contains some basic settings that apply to all pages:

- Name
- Description
- Page Type

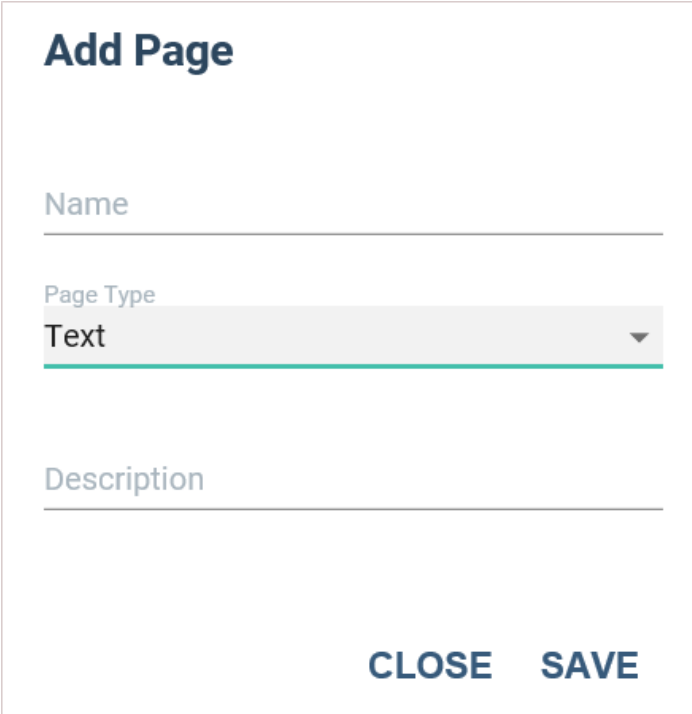

Add Page

Name


Page Type

Text

Description

CLOSE **SAVE**

Other configuration options become accessible in **Edit mode**.

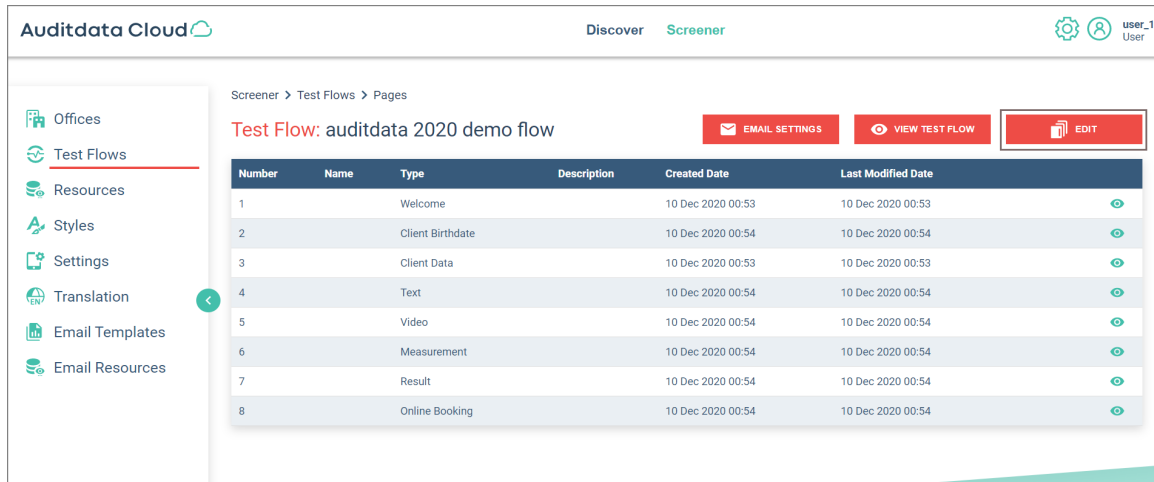

WELCOME PAGE

The **Welcome page** is a mandatory page that must always come first. A Test Flow cannot be published without this page.

Edit Welcome page

GENERAL
MARGINS

Name
Welcome
Description

Background Image
▼

Welcome Page Text

Next Button Caption
Continue

CLOSE SAVE

After the Welcome page has been added, you can further adjust the following:

- *Background image*: when you click the corresponding drop-down, you will see a number of options to choose from
- *Welcome Page text*
- *Next Button caption*
- *Margins* for Title Position

CLIENT DATA PAGE

The **Client Data page** contains more detailed information about the client: for instance, their name, email and phone number. Information fields can be assigned a mandatory status.

Client Data page includes the following tabs:

- General
- Texts
- Fields
- Agreement
- Margins

On the **General tab**, you can set the page's:

- Name
- Description
- Background Image

The **Texts tab** allows configuring:

- The Header text
- The Next Button caption

The **Fields tab** includes fields that can be shown on the page, as well as a personal title list that will be able as a drop-down option in your Listo app.

Edit Client Data page

GENERAL
TEXTS
FIELDS
AGREEMENT
MARGINS

	Show	Mandatory		
Title	<input checked="" type="checkbox"/>	<input type="checkbox"/>	Title list	+
Title	<input checked="" type="checkbox"/>	<input type="checkbox"/>	Mr	×
First Name	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	Mrs	×
First Name	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	Ms	×
Last Name	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>		
Last Name	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>		
Email	<input checked="" type="checkbox"/>	<input type="checkbox"/>		
Email	<input checked="" type="checkbox"/>	<input type="checkbox"/>		

CLOSE
SAVE

The **Agreement tab** allows you to configure Customer Agreement in compliance with GDPR.

Edit Client Data page

GENERAL
TEXTS
FIELDS
AGREEMENT
MARGINS

Is Consent Visible

Consent

Is Marketing Statement Enabled

Marketing Statement text

Marketing statement opt in

Marketing Statement font size

12

Link to Policy

Policy text

Is Privacy Statement Enabled

Privacy Statement text

Privacy statement opt in

Privacy Statement font size

12

CLOSE
SAVE

On this tab, you can:

- Make *Customer Consent* visible
- Specify the link to the *Privacy Policy*
- Add a *Privacy Policy* text to the Client Data page
- Enable *Marketing* and *Privacy Statements*

The **Margins tab** allows adjusting the Title or Datepicker positions.

Edit Client Data page

GENERAL
TEXTS
FIELDS
AGREEMENT
MARGINS

List Of Blocks

Title Block Position ▼

Top

50

Left

0

Bottom

0

Right

0

CLOSE
SAVE

TEXT PAGE

The **Text page** can be used to add any information that can be helpful during an audiometric test. You can also configure the position of the text on the **Margins tab**.

The screenshot shows a form titled "Edit Let's get started page" with two tabs: "GENERAL" (selected) and "MARGINS". The form contains the following fields:

- Name:** A text input field.
- Description:** A text input field containing the text "If this is your first hearing test, consider the following|".
- Background Image:** A dropdown menu.
- Next Button Caption:** A text input field containing the text "Continue".

At the bottom right of the form, there are two buttons: "CLOSE" and "SAVE".

VIDEO PAGE

The **Video page** can be used to add an introductory video preparing a client for the test, a post-test video explanation of the hearing assessment results or recommendations. Its settings allow:

- Customizing the *Next Button caption* and *Page Header*
- Adding selected background image, video and text
- Setting the margins for the title, text and video blocks on the **Margins tab**

For recommended video and image specifications, see [Section 1: Setup and Get Started > Resources > Media File Specifications](#)

The screenshot shows a form titled "Edit Video page" with two tabs: "GENERAL" (selected) and "MARGINS". The form contains the following fields:

- Name:** A text input field.
- Description:** A text input field.
- Background Image:** A dropdown menu.
- Text:** A text input field.
- Header:** A text input field containing the text "Introduction".
- Next Button Caption:** A text input field containing the text "Continue".
- Select Video:** A dropdown menu containing the text "auditdata_screener_intro.mp4".

At the bottom right of the form, there are two buttons: "CLOSE" and "SAVE".

MEASUREMENT PAGE

The **Measurement page** has a number of settings that can be accessed from its tabs:

- General
- Texts
- Frequency
- Test Settings

On the **General tab**, you can set the background image for the whole page, as well as additional specific images for the left and right ear measurements.

The **Texts tab** contains settings for different types of text that may appear on the page.

Edit Measurement page

GENERAL **TEXTS** FREQUENCY TEST SETTINGS MARGINS

Start Header
Before you start the test

1 Make sure you are in a quiet area for best

Start Explanation

Left Header
Now let's check your left ear

2 When you think you hear a tone, press the

Left Explanation

Right Header
We will now check your right ear

3 When you think you hear a tone, press the

Right Explanation

Start Button Caption
Start

4 Response

Response Button Caption

CLOSE SAVE

Here you can define:

- The *Start Measurement Header and Explanation*
- The *Left Measurement and Header*
- The *Right Measurement and Header*
- *Captions for the Start and the Response buttons*

The **Frequency tab** makes it possible to determine a set of frequencies for a hearing exam.

Edit Measurement page

GENERAL TEXTS **FREQUENCY** TEST SETTINGS MARGINS

Frequency, Hz

250

500

1000

2000

4000

6000

8000

CLOSE SAVE

The **Test Settings** tab allows users to specify the key parameters for the measurement procedure

Edit Measurement page

GENERAL TEXTS FREQUENCY **TEST SETTINGS** MARGINS

Threshold Determination
2 out of 3 ascents

Stimulus Type
Pulse

Test, dB
Min: 20 Start: 30 Max: 80

Initial Ascending Step, dB
20

Ascending Step, dB
5

Initial Descending Step, dB
15

Descending Step, dB
10

Is Fixed-Length Tone

CLOSE SAVE

Here you can configure:

- *Threshold determination* setting that specifies how a hearing threshold is found
- *Stimulus Type*: choose from the available Tone, Pulse, Warble or PulseWarble options
- *Volume* of the sounds presented during a hearing test
- *Initial ascending step*: first increase in the intensity of the sound
- *Ascending step*: all subsequent increases in sound intensity
- *Initial descending step*: first decrease in the intensity of the sound
- *Descending step*: all subsequent decreases in sound intensity
- *Fixed-length tone* option, allowing users to keep tone duration the same or make it varied

The **Margins tab** contains margin settings for the title, text, button and progress bar blocks.

Edit Measurement page

GENERAL TEXTS FREQUENCY TEST SETTINGS **MARGINS**

Title Block Position

Text Block Position

Button Block Position

Progress Bar Block Position

Bottom
0

Right
0

CLOSE SAVE

RESULT PAGE

The **Result page** has a number of settings that can be accessed from its tabs:

- General
- Texts
- Fields
- Calculation
- Margins

On the **General tab**, you can set the background image for the whole page, as well as overlay colors for normal hearing, some hearing loss and significant hearing loss audiograms.

The **Texts tab** contains settings for different types of text that may appear on the page, specifically the Header text and the Next Button caption.

The **Fields tab** allows users to:

- Add videos for various assessment outcomes: normal hearing, some hearing loss and a significant hearing impairment
- Configure the text to explain the results of the hearing assessment
- Add custom text to the audiogram's measurements for the left and right ears respectively

Edit Result page

GENERAL TEXTS **FIELDS** CALCULATION MARGINS

Normal Hearing Video
auditdata_screener_noloss.mp4

Some Hearing Loss Video
auditdata_screener_someloss.mp4

Significant Hearing Loss Video
auditdata_screener_significantloss.mp4

Left Ear Text
Left Ear

Normal Hearing Text
Normal hearing

Significant Hearing Loss Text
Significant hearing loss

Normal Hearing Video Sound Level (%)
50

Some Hearing Loss Video Sound Level (%)
50

Significant Hearing Loss Video Sound Level (%)
50

Right Ear Text
Right Ear

Some Hearing Loss Text
Some hearing loss

CLOSE SAVE

The **Calculation tab** makes it possible to designate the corresponding values to *Some* or *Significant hearing loss* ranges, as well as determine the weighting of particular frequencies used to calculate the PTA.

Edit Result page

GENERAL TEXTS FIELDS **CALCULATION** MARGINS

Calculation Method
PTA

Some Hearing Loss Value
30

Significant Hearing Loss Value
50

Frequency, Hz	Test	Weighting	Some HL, dB	Significant HL, dB
250	<input type="checkbox"/>	1.00	40.00	60.00
500	<input type="checkbox"/>	1.00	40.00	60.00
1000	<input checked="" type="checkbox"/>	1.00	40.00	60.00
2000	<input checked="" type="checkbox"/>	1.00	40.00	60.00
4000	<input checked="" type="checkbox"/>	1.00	40.00	60.00

CLOSE SAVE

The **Margins tab** includes margin settings for the title, chart, result, video and text blocks.

Edit Result page

GENERAL
TEXTS
FIELDS
CALCULATION
MARGINS

- Title Block Position
- Chart Block Position
- Result Block Position
- Video Block Position
- Left Ear Text Block Position
- Right Ear Text Block Position

Bottom

Right

CLOSE SAVE

CLIENT BIRTHDATE PAGE

The **Client Birthdate page** is the page where clients input their date of birth. Its settings are identical to those of the Welcome page and can be modified in the same way.

Edit Client Birthdate page

GENERAL
MARGINS

Name

Background Image

Header

Next Button Caption

Description

CLOSE SAVE


This page also has the **Margins** tab with margin settings for the title and datepicker blocks.

3 Run Hearing Tests

Having created a [Test Flow](#) in the Auditdata Cloud, you can start using the Listo app to conduct hearing assessment for your clients.

To do so:

1. Tap the **Listo icon** to open the app

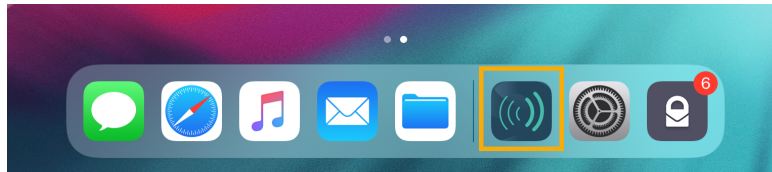

2. On the **Setup Application page**, select the following:
 - Office
 - Transducer
 - Test Flow
 - Noise Level

Logout

SETUP APPLICATION

Selected office: Store Name

Choose an Office

Choose a Transducer

DD65

DT770Pro

HD622F

K72

Choose a Flow

Auditdata 2020 Demo Flow

Choose a Noise Level

Silent


Some Noise

A Lot of Noise

Apply the Flow

Version 1.3.2 License: 577yeMmzolsW

3. To change the settings in the Listo app, tap the **invisible button** in the upper right corner. You will need a password to proceed.

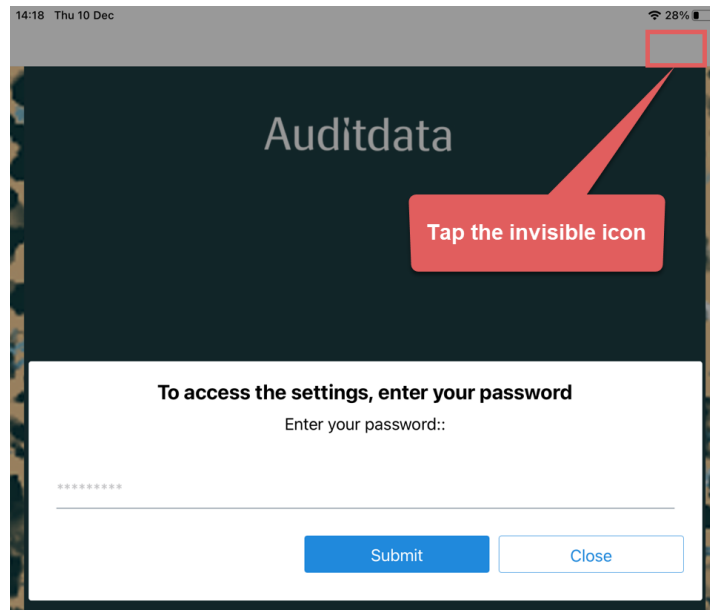

There are two types of passwords:

- Setup settings password
- Noise correction password

Use any of the above to either change the whole application setup, or apply a different noise level to the selected Test Flow.

Both passwords are configured in the Auditdata Cloud. For more information, go to [Section 1: Set Up and Get Started > Settings](#)


4. Once you are done with configuration, you can use the selected Test Flow for hearing assessment.

Depending on the Test Flow settings, you will go through 3 to 8 pages.


Below is a sample Test Flow. The Test Flow you will be using might have a slightly different structure.

5. Follow the steps of the Test Flow you have chosen.

The **Welcome page**: regardless of the settings, you will always see this page. Just tap the button to continue.

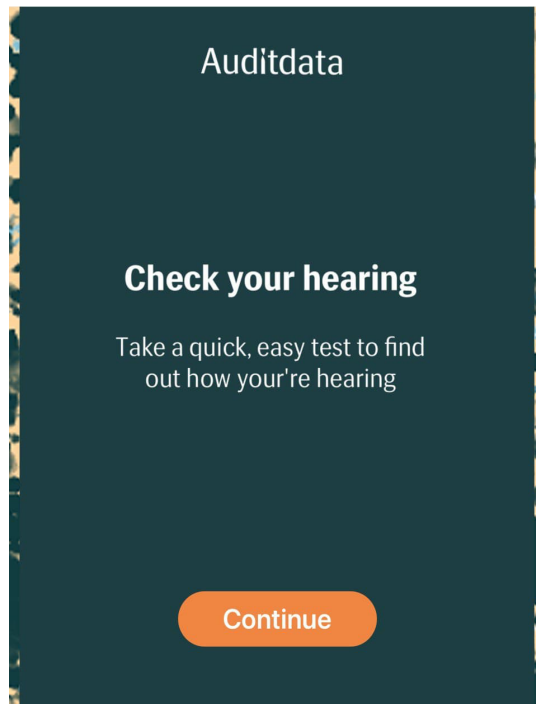


The **Client Birthdate page** is optional and its placement is not fixed. This page may come right after the Welcome page or somewhere in the middle of the flow.

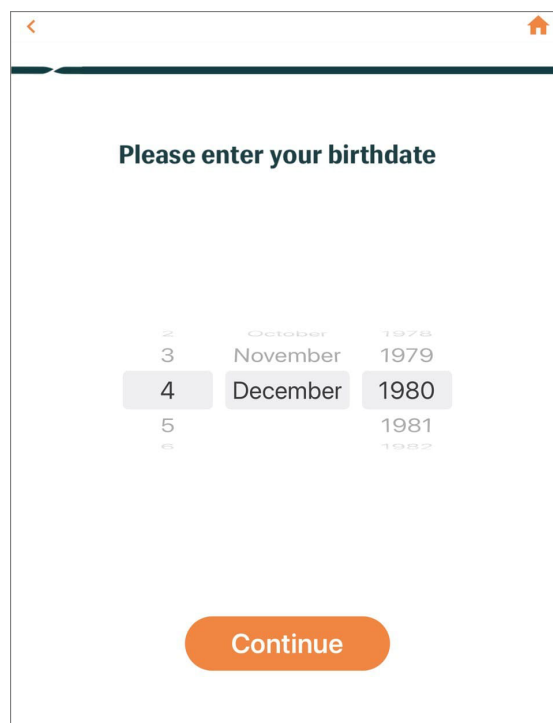


The **Client Data** page is also optional.

- If it appears in the flow, complete the required personal information fields and, if needed, select *Privacy* and/or *Marketing statement opt-in(s)* to continue.
- When all the required information is provided, the Continue button becomes active and one may carry on with the test.

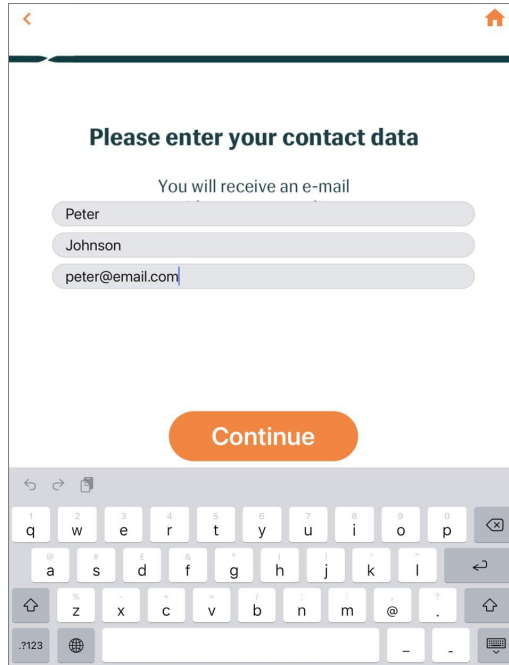


A screenshot of a mobile application screen titled "Please enter your contact data". The screen displays a form with three input fields: "Name" (containing "Peter Johnson"), "Email" (containing "peter@email.com"), and a "Continue" button. Above the form, it says "You will receive an e-mail". The keyboard is visible at the bottom of the screen.


In this example, the **Video** page serves the purpose of familiarizing the clients with the hearing test instructions. It may be used for a different reason.

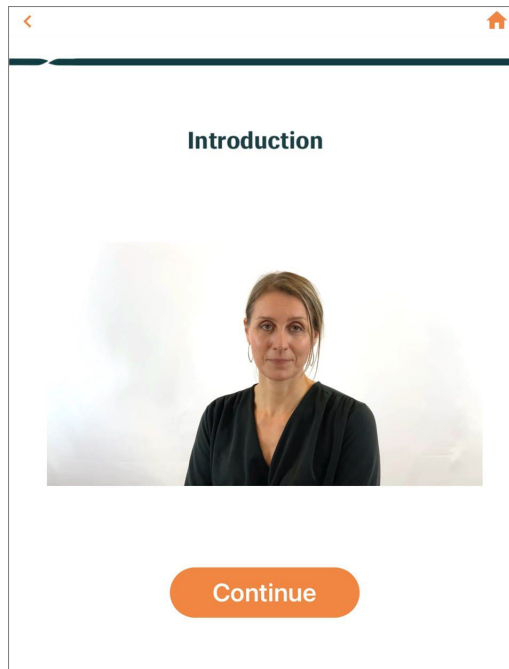


The **Text page** is optional and may be configured to contain any other information relevant to the person taking the test.

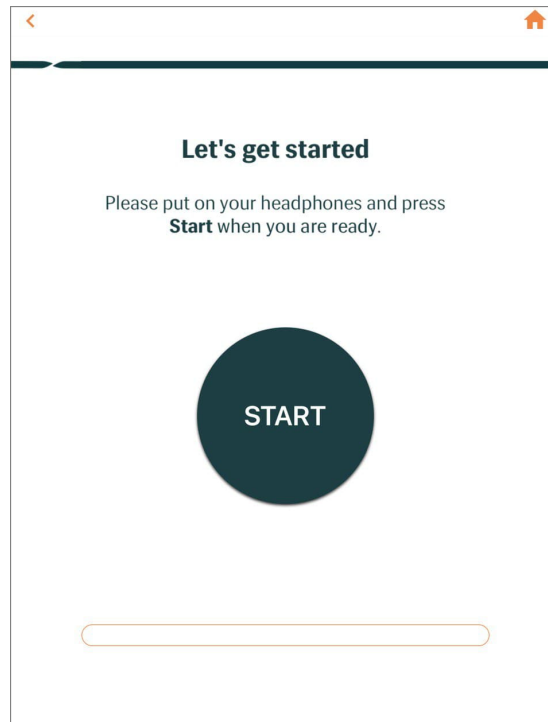

The **Measurement page** is where the hearing level is tested.

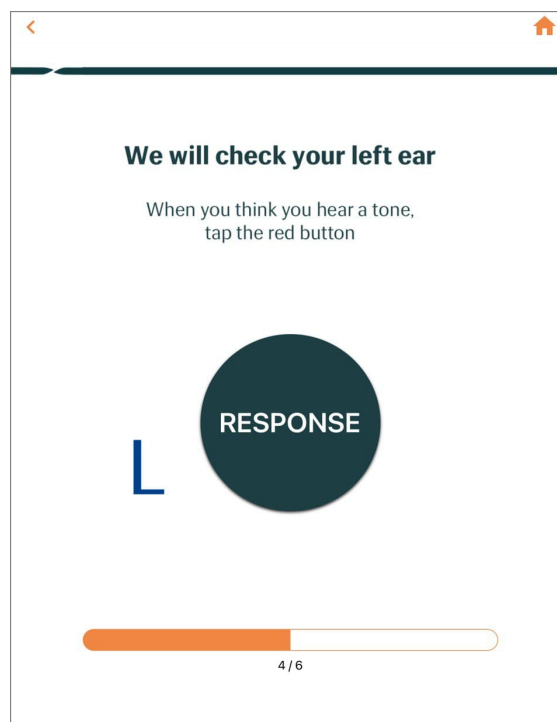

The **Result page** will contain a graph with the plotted hearing threshold values for the right and the left ears. Optionally, it may have a video explaining the results or contain any further instructions. Clients need to tap the button to either finish the test or continue if the appointment booking option is included in the flow.

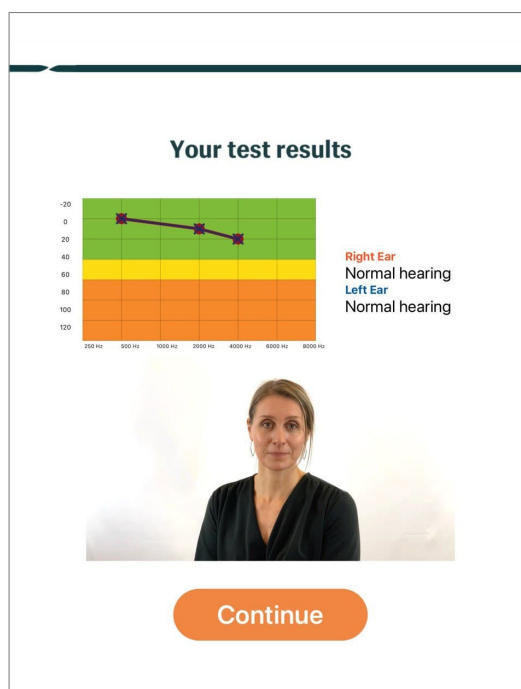


4 Get Listo Reports

You can look up and download Listo usage and customer data reports from your Auditdata Cloud.

Prerequisites: permissions are required to view Listo reports. As a user you may have either of the following: permission to view all report types or permission to access the usage report only.

Note: if reports are hidden from your user view, contact your system administrator.

1. To access Listo reports, log into **Auditdata Cloud** > click **Screener** > select **Reports**

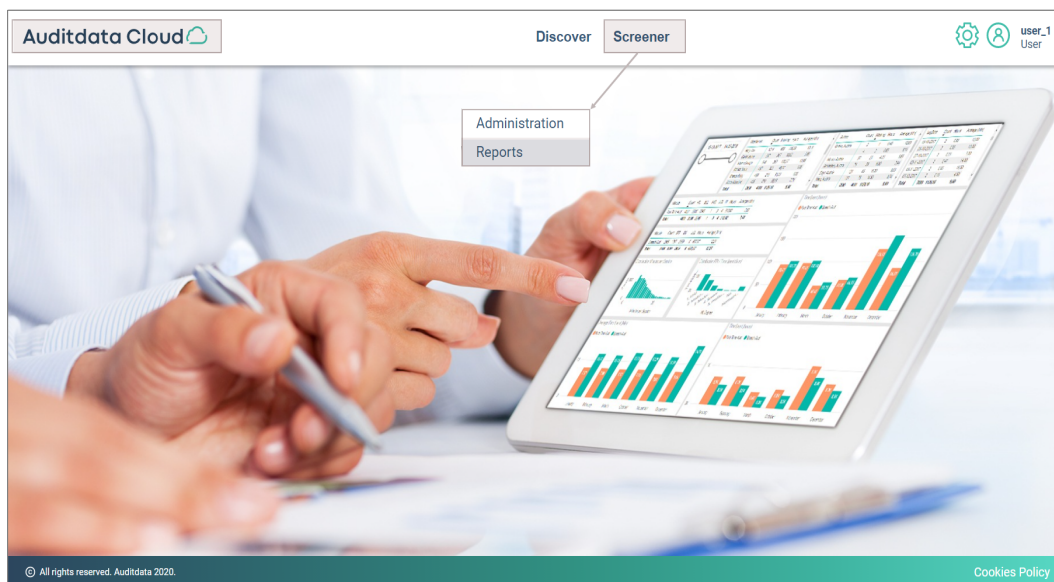

2. You can search **Usage** or **Customer Data** reports by *date* and/or *office* and download them to a specified location.

