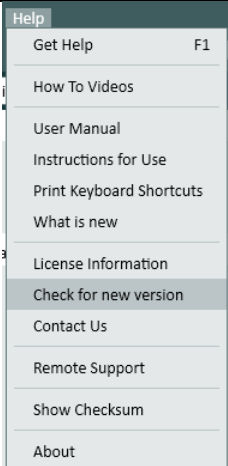


Auditdata

Support Bulletin

Support Bulletin Details			
Title:	Support bulletin #06	Support Bulletin #	DN02551/06
Product:	2000 Audiometers, 2000-1 Fitting Unit (Primus, Unity 3, Measure, Unity 4)	Date:	12/05/2026
Reason for Action:	Moderate security vulnerability	Author:	Samantha Dixon
Information Classification:	Public	Status:	Approved

Description	
Issue:	
<p>In April 2026, The National Vulnerability Database published details of a security vulnerability. This issue fails to sanitize forbidden characters and the identity field that may carry attacker-influenced data. This causes a silent loss of the affected log event. An attacker who can influence any of these fields can exploit this to suppress individual log records, impairing audit trails and detection of malicious activity. https://nvd.nist.gov/vuln/detail/CVE-2026-40021</p> <p>This vulnerability is present in Measure, Unity and Otometrics Measure software versions 6.4.1 and earlier.</p>	
Solution:	
<p>Upgrade Measure, Unity or Otometrics Measure software to version 6.5.0 or later where this issue is fixed. The release is targeted for end of May 2026.</p> <p>When released, the new Measure software will be available to download here: https://support.auditdata.com/portal/en/kb/articles/file-downloading-primus-software</p> <p>Users may also click Help - Check for new version directly in the application.</p>	
	
Support and Assistance:	
<p>If you need assistance with Auditdata software or have any concerns, our support team is ready to help. You can contact us by sending an email to: support@auditdata.com</p>	